

## USER MANUAL

### Humalog KwikPen 200 units/ml solution for injection in a pre-filled pen

insulin lispro



**PLEASE READ THIS USER MANUAL BEFORE USE**



**INJECT THE MEDICINE ONLY WITH THIS PEN OR  
SEVERE OVERDOSE CAN RESULT**

Read the User Manual before you start taking **Humalog KwikPen 200 units/ml** solution for injection and each time you get another **Humalog KwikPen 200 units/ml**. There may be new information. This information does not take the place of talking to your healthcare professional about your medical condition or your treatment.

**Humalog KwikPen 200 units/ml** ("Pen") is a disposable pre-filled pen containing 3 ml (600 units, 200 units/ml) of insulin lispro solution for injection. You can give yourself multiple doses using one Pen. The Pen dials 1 unit at a time. You can inject from 1 to 60 units in a single injection. **If your dose is more than 60 units, you will need to give yourself more than one injection.** The plunger only moves a little with each injection, and you may not notice that it moves. The plunger will only reach the end of the cartridge when you have used all 600 units in the Pen.


**This pen is designed to allow you to give more doses than other pens you may have used in the past. Dial your usual dose as instructed by your healthcare professional.**

**Humalog KwikPen is available in two strengths, 100 units/ml and 200 units/ml. Inject Humalog KwikPen 200 units/ml ONLY with your Pen. DO NOT transfer insulin from your Pen to another insulin delivery device. Syringes and insulin pumps will not dose 200 units/ml insulin correctly. A severe overdose can result, causing very low blood sugar which may put your life in danger.**


**Do not share your Pen with other people, even if the needle has been changed. Do not reuse or share needles with other people. You may give an infection to them or get an infection from them.**

This Pen is not recommended for use by the blind or visually impaired without the help of someone trained to use the Pen.

### KwikPen Parts

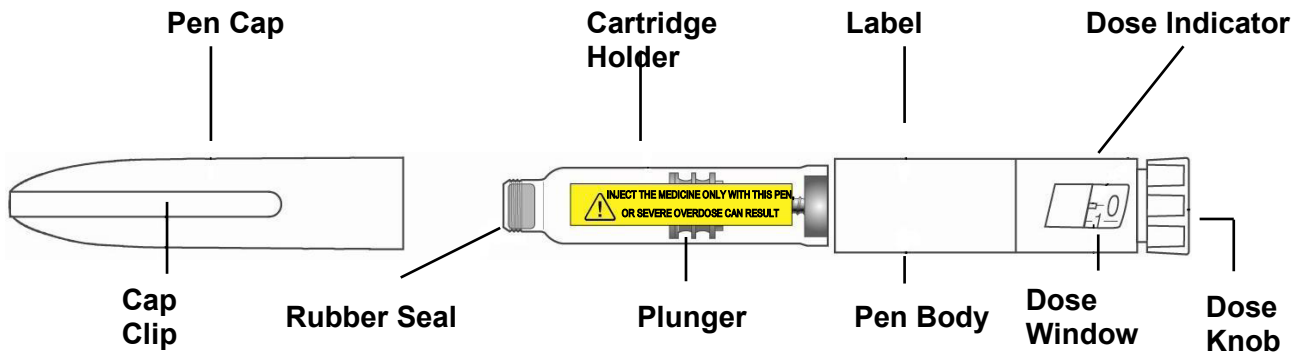

### Pen Needle Parts (Needles Not Included)

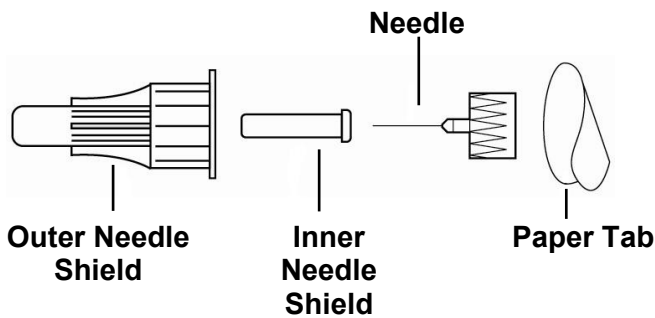

### Dose Knob with burgundy ring



### How to recognize your Humalog KwikPen 200 units/ml:

- Pen color: Dark gray
- Dose Knob: Dark gray with burgundy ring on the end
- Labels: Burgundy, "200 units/ml" in yellow box.  
Yellow warning on cartridge holder

### Supplies needed to give your injection:

- Humalog KwikPen 200 units/ml
- KwikPen compatible Needle (BD [Becton, Dickinson and Company] Pen Needles recommended)
- Swab

Needles and swab are not included.

### Preparing your Pen

M HUMAKPN 200 F 03

- Wash your hands with soap and water.
- Check the Pen to make sure you are taking the right type of insulin. This is especially important if you use more than 1 type of insulin.
- **Do not** use your Pen past the expiration date printed on the Label or for more than 28 days after you first start using the Pen.
- Always use a **new Needle** for each injection to help prevent infections and blocked Needles.

---



**Step 1:**

Pull the Pen Cap straight off.

- **Do not** remove the Pen Label.

Wipe the Rubber Seal with a swab.

**Humalog KwikPen 200 units/ml** solution for injection should look clear and colorless. **Do not** use if it is cloudy, colored, or has particles or clumps in it.

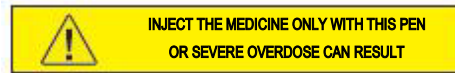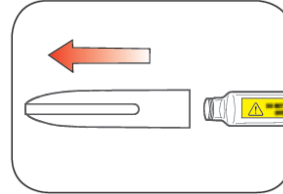


---

**Step 2:**

Select a new Needle.


Pull off the Paper Tab from the Outer Needle Shield.

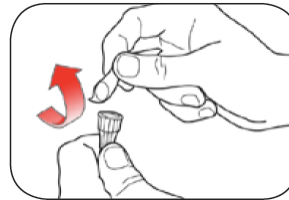

---

**Step 3:**

Push the capped Needle straight onto the Pen and twist the Needle on until it is tight.

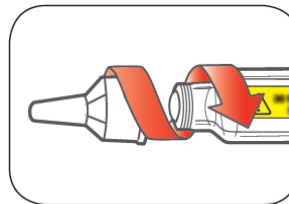


---

**Step 4:**

Pull off the Outer Needle Shield. **Do not** throw it away.

Pull off the Inner Needle Shield and throw it away.

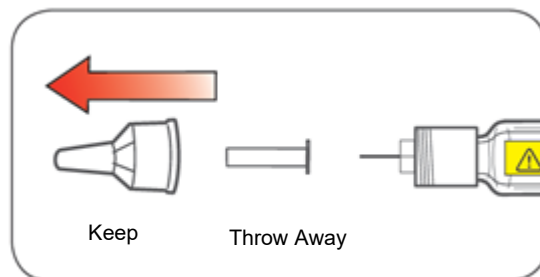

## Priming your Pen


### Prime before each injection.

- Priming your Pen means removing the air from the Needle and Cartridge that may collect during normal use and ensures that the Pen is working correctly.
- If you do not prime before each injection, you may get too much or too little insulin.

---

#### Step 5:

To prime your Pen, turn the Dose Knob to **select 2 units**.

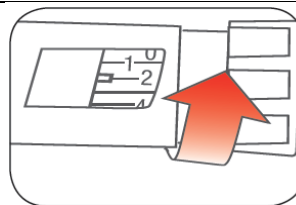

---

#### Step 6:

Hold your Pen with the Needle pointing up. Tap the Cartridge Holder gently to collect air bubbles at the top.





---

#### Step 7:

Continue holding your Pen with Needle pointing up. Push the Dose Knob in until it stops, and "0" is seen in the Dose Window. Hold the Dose Knob in and **count to 5 slowly**.

- You should see insulin at the tip of the Needle.
  - If you **do not** see insulin, repeat the priming steps, but not more than 8 times.
  - If you **still do not** see insulin, change the Needle and repeat the priming steps.

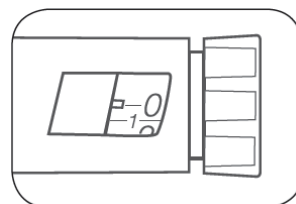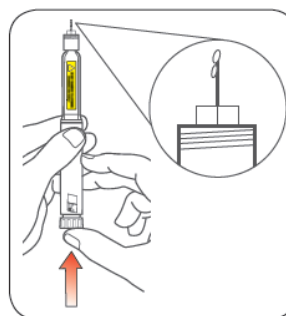

Small air bubbles are normal and will not affect your dose.

---

## Selecting your dose


This pen has been designed to deliver the dose that is shown in the window. Dial your usual dose as instructed by your healthcare professional.

- You can give from 1 to 60 units in a single injection.
- If your dose is more than 60 units, you will need to give more than one injection.
  - If you need help deciding how to divide up your dose, ask your healthcare professional.
  - You should use a new Needle for each injection and repeat the priming step.


---

### Step 8:


Turn the Dose Knob to select the number of units you need to inject. The Dose Indicator should line up with your dose.

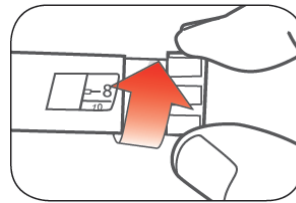

- The Pen dials 1 unit at a time.
- The Dose Knob clicks as you turn it.
- DO NOT dial your dose by counting the clicks because you may dial the wrong dose.
- The dose can be corrected by turning the Dose Knob in either direction until the correct dose lines up with the Dose Indicator.
- The **even** numbers are printed on the dial.
- The **odd** numbers, after the number 1, are shown as full lines.

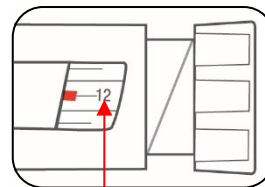

(Example: 12 units shown in the Dose Window)

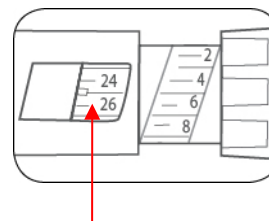

(Example: 25 units shown in the Dose Window)

**Always check the number in the Dose Window to make sure you have dialed the correct dose.**

---

M HUMAKPN 200 F 03

- The Pen will not let you dial more than the number of units left in the Pen.
- If you need to inject more than the number of units left in the Pen, you may either:
  - inject the amount left in your Pen and then use a new Pen to give the rest of your dose, **or**
  - get a new Pen and inject the full dose.

### Giving your injection

- Inject your insulin as your healthcare professional has shown you.
- Change (rotate) your injection site for each injection.
- **Do not** try to change your dose while injecting.


---

#### Step 9:

Choose your injection site.

**Humalog KwikPen 200 units/ml** solution for injection is injected under the skin (subcutaneously) of your stomach area, buttocks, upper legs or upper arms.

Wipe your skin with a swab, and let your skin dry before you inject your dose.

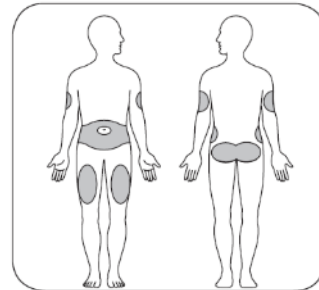

---


#### Step 10:

Insert the Needle into your skin.

Push the Dose Knob all the way in.



Continue to hold the Dose Knob in and **slowly count to 5** before removing the Needle.

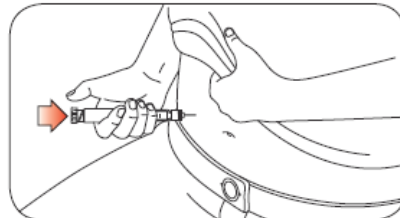


Do not try to inject your insulin by turning the Dose Knob. You will **NOT** receive your insulin by turning the Dose Knob.

---

**Step 11:**

Pull the Needle out of your skin.

- A drop of insulin at the Needle tip is normal. It will not affect your dose.
- Check the number in the Dose Window. If you see “0” in the Dose Window, you have received the full amount you dialed.
- If you do not see “0” in the Dose Window, do not redial. Insert the needle into your skin and finish your injection.
- If you **still** do not think you received the full amount you dialed for your injection, **do not start over or repeat that injection**. Monitor your blood glucose as instructed by your healthcare professional.

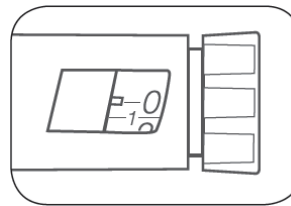

The plunger only moves a little with each injection and you may not notice that it moves.

If you see blood after you take the Needle out of your skin, press the injection site lightly with a piece of gauze or a swab. **Do not** rub the area.


---

**After your injection**

---


**Step 12:**

Carefully replace the Outer Needle Shield.

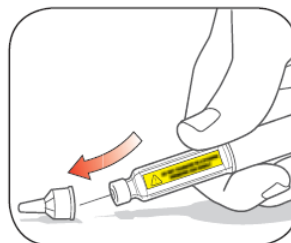

**Step 13:**

Unscrew the capped Needle and dispose of it as described below (see **Disposing of Pens and Needles** section).

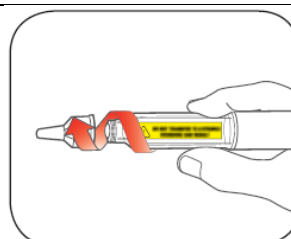


Do not store the Pen with the Needle attached to prevent leaking, blocking the Needle, and air from entering the Pen.

---

**Step 14:**

Replace the Pen Cap by lining up the Cap Clip with the Dose Indicator and pushing straight on.

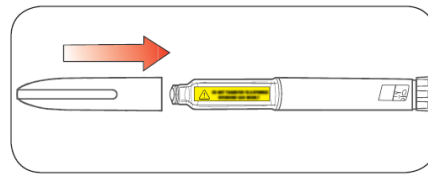

---

**Disposing of Pens and Needles**

- Put used Needles in a sharps container or hard plastic container with a secure lid. Do not throw needles directly into your household trash.
- Do not recycle the filled sharps container.
- Ask your healthcare professional about options to dispose of the pen and the sharps container properly.
- The directions regarding needle handling are not intended to replace local, healthcare professional or institutional policies.

**Storing your Pen**

**Unused Pens**

- Store unused Pens in the refrigerator at 2°C to 8°C.
- **Do not** freeze **Humalog KwikPen 200 units/ml** solution for injection. **Do not** use if it has been frozen.
- Unused Pens may be used until the expiration date printed on the Label, if the Pen has been kept in the refrigerator.

**In-use Pen**

- Store the Pen you are currently using at room temperature (below 30°C) and away from dust, food and liquids, heat and light.
- Throw away the Pen you are using after 28 days, even if it still has insulin left in it.

**General information about the safe and effective use of your Pen**

- **Keep your Pen and Needles out of the sight and reach of children.**
- **Do not** use your Pen if any part looks broken or damaged.
- Always carry an extra Pen in case yours is lost or damaged.

**Troubleshooting**

- If you cannot remove the Pen Cap, gently twist the cap back and forth, and then pull the cap straight off.
- If the Dose Knob is hard to push;



M HUMAKPN 200 F 03

- Pushing the Dose Knob more slowly will make it easier to inject.
- Your Needle may be blocked. Put on a new Needle and prime the Pen.
- You may have dust, food, or liquid inside the Pen. Throw the Pen away and get a new Pen. You may need to get a prescription from your healthcare professional.
- **Do not transfer insulin from the Pen to a syringe or an insulin pump. Severe overdose can result.**

If you have any questions or problems with your **Humalog KwikPen 200 units/ml**, call your healthcare professional for help.

Humalog® and Humalog® KwikPen™ are trademarks of Eli Lilly and Company.

<b>Humalog KwikPen</b> meets the current dose accuracy and functional requirements of ISO 11608-1.
--

Copyright © 2022, Eli Lilly and Company. All rights reserved.

**Manufacturer:** Lilly France S.A.S., Fegersheim, France

**License holder:** Eli Lilly Israel Ltd., 4 HaSheizaf St., P.O.Box 4246, Ra'anana

Revised in June 2022 according to MOH guidelines.

M HUMAKP 200 F 03

The Lilly logo is written in a red, cursive script font.