

## Teriparatide Teva


### Solution for injection in a pre-filled pen

**Important: First read the package leaflet that comes inside your Teriparatide carton.**


**Before you use your Teriparatide Teva Pen, please read the entire User Manual. Follow the directions carefully when using the pen.**

**The Teriparatide Teva Pen contains 28 days of medicine. Throw away the pen after 28 days, even if it is not completely empty. Do not inject more than one dose of Teriparatide Teva in the same day. Do not transfer Teriparatide Teva to a syringe.**

#### Teriparatide Teva Pen Parts\*:

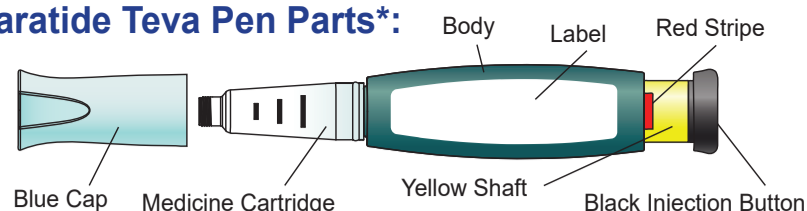


\* **Needles not included.**  
29G to 31G needles can be used. Ask your doctor or pharmacist which needle size and length are best for you.  
**Always use a new needle for each injection.**

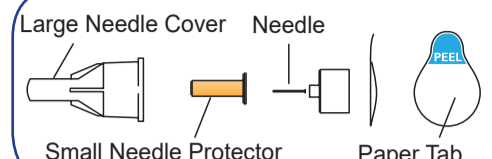


**Wash your hands before every injection. Prepare the injection site as your doctor or pharmacist instructed.**

### 1 Pull off blue cap

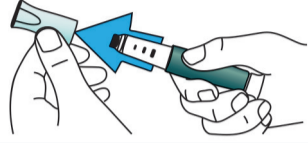


Check the Teriparatide Teva Pen label to make sure you have the right medicine and that it has not expired. Do not use the pen if the pen looks damaged, the medicine in the cartridge is not clear and colorless, or if it has particles in it.


### 2 Attach new needle

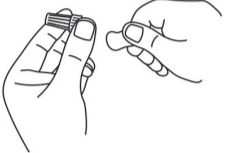


Pull off paper tab.

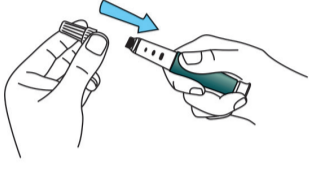

Push needle **straight** onto medicine cartridge.

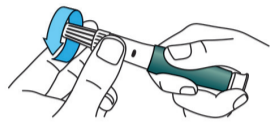


Screw on needle clockwise until it stops (3 to 5 clockwise turns). Make sure the needle is firmly attached.

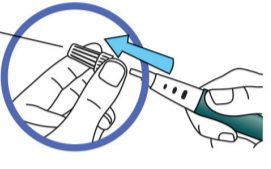


Pull off large needle cover and **save it**.

### 3 Set dose


**Pull out black injection button until it stops and the red stripe shows.** If you cannot pull out the black injection button see 'Troubleshooting', Problem E, in this manual.

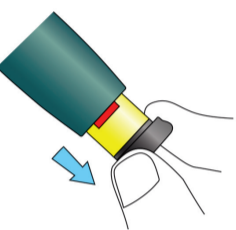


Check to make sure red stripe shows.

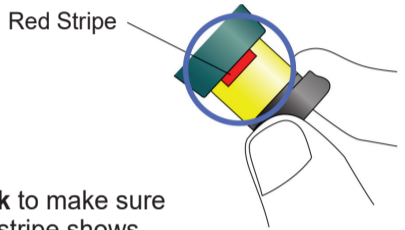


Pull off small needle protector and throw away.

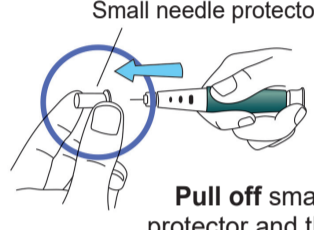

### 4 Inject dose

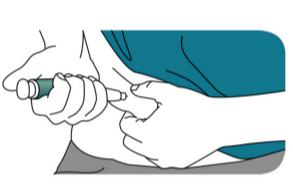


Gently hold a fold of skin on your thigh or abdomen and insert needle straight into skin.




**Push in black injection button until it stops. Keep the button pushed in and count to 5 slowly. You must wait until the count of 5 to make sure you receive the correct dose.** Then pull the needle from skin.

### 5 Confirm dose




**After injecting the dose:** Once the needle is removed from the skin, take your thumb off the black injection button. **Check** to make sure the black injection button is all the way in. If the yellow shaft does not show, you have finished the injection steps the right way.

**IMPORTANT!**




You should not see any of the yellow shaft. If you do and have already injected the medicine, do not inject yourself a second time on the same day. Instead, **you MUST reset the Teriparatide Teva Pen** (see 'Troubleshooting', Problem A, in this manual).


### 6 Remove needle

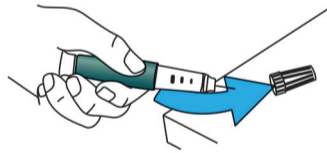


Put the large needle cover on the needle. Do not try to put the needle cover back on with your hands.

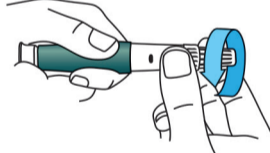

Unscrew the covered needle all the way by giving the large needle cover 3 to 5 counter-clockwise turns.



Pull off needle and throw away as directed by your doctor or pharmacist.

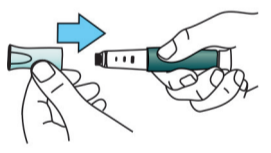

Push blue cap back on. Right after use, place Teriparatide Teva Pen in the refrigerator.

**For more information, or if you have any questions, see the section 'Troubleshooting' in this manual.**

## Troubleshooting

Problem	Solution
<b>A. The yellow shaft is still showing after I push in the black injection button. How do I reset my Teriparatide Teva Pen?</b>	<p>This problem can be prevented by using a <b>NEW</b> needle with each injection, making sure that the needle is attached properly, and pushing the black injection button all the way in. To reset the Teriparatide Teva Pen, follow the steps below.</p> <ol style="list-style-type: none"> <li>1. If you have already injected, <b>DO NOT</b> inject yourself a second time on the same day.</li> <li>2. Remove the needle.</li> <li>3. Repeat <b>Step 2</b> 'Attach new needle' and <b>Step 3</b> 'Set Dose' in the User Manual.</li> </ol>
<b>B. How can I tell if my Teriparatide Teva Pen works properly?</b>	The black injection button should be pushed all the way in to show that the full dose of medicine has been injected from the Teriparatide Teva Pen. Use a new needle every time you inject to be sure your Teriparatide Teva Pen will work properly.
<b>C. I can see an air bubble in my Teriparatide Teva Pen.</b>	A small air bubble will not affect your dose and it will not harm you. You can continue to take your dose as usual.
<b>D. I cannot get the needle off.</b>	<ol style="list-style-type: none"> <li>1. Put the large needle cover on the needle.</li> <li>2. Use the large needle cover to unscrew the needle.</li> <li>3. Unscrew the needle all the way by giving the large needle cover 3 to 5 counter-clockwise turns.</li> <li>4. If you still cannot get the needle off, ask someone to help you.</li> </ol>
<b>E. What should I do if I have difficulty pulling out the black injection button?</b>	Contact your doctor. You may need to use a new Teriparatide Teva Pen. When the black injection button cannot be pulled out, this means there is not enough medicine in your Teriparatide Teva Pen for another dose. You may still see some medicine left in the cartridge.

#### Cleaning and Storage:

##### Cleaning Your Teriparatide Teva Pen:

- Wipe the outside of the Teriparatide Teva Pen with a damp cloth.
- Do not place the Teriparatide Teva Pen in water, or wash or clean it with any liquid.

##### Storing Your Teriparatide Teva Pen:

- After each use, refrigerate the Teriparatide Teva Pen right away. Read and follow the storage instructions in the patient leaflet section 5 - 'How to store the medicine?'
- Do not store the Teriparatide Teva Pen with the needle attached. Doing this may cause air bubbles to form in the medicine cartridge.
- Store the Teriparatide Teva Pen with the blue cap on.
- Do not freeze Teriparatide Teva Pen. If the pen has been frozen, throw it away and use a new pen.
- If the Teriparatide Teva Pen has been left out of the refrigerator, do not throw the pen away. Place the pen back in the refrigerator and contact your doctor or pharmacist.

#### Other Important Notes:

- The Teriparatide Teva Pen contains 28 days of medicine.
- Do not transfer the medicine to a syringe. This may result in you taking the wrong dose of medicine.
- Write down the date of your first injection in your diary.
- Read and follow the instructions in this manual so that you use your Teriparatide Teva Pen the right way.
- Check the Teriparatide Teva Pen label to make sure you have the right medicine and that it has not expired.

- Do not use the Teriparatide Teva Pen if it looks damaged. Look at the medicine in the cartridge. If the solution is not clear and colorless, or if it has particles, do not use the pen. Contact your doctor or pharmacist if you notice any of these signs.
- Use a new needle for each injection.
- During the injection, you may hear one or more clicks – this is normal.
- The Teriparatide Teva Pen is not recommended for use by the blind or by those who have vision problems without help from a person trained in the proper use of the device.
- Keep your Teriparatide Teva Pen and needles out of the reach and sight of children.

#### Disposal of Pen Needles and the Teriparatide Teva Pen:

- Before throwing away the pen, be sure to remove the pen needle.
- Throw away your Teriparatide Teva Pen and used needles as directed by your doctor or pharmacist.

#### Dispose of the Teriparatide Teva Pen 28 days after first use.

1<sup>st</sup> use date ..... / ..... / .....  
Throw away after ..... / ..... / .....

Registration number of the medicine in the Ministry of Health's National Drug Registry:  
163-24-35333

Manufacturer and registration holder's name and address:  
Teva Israel Ltd., 124 Devorah Haneviah Street, Tel Aviv 6944020.

Revised in September 2022 according to MOH guidelines.

DOR-Ter-IFU-1122-09

