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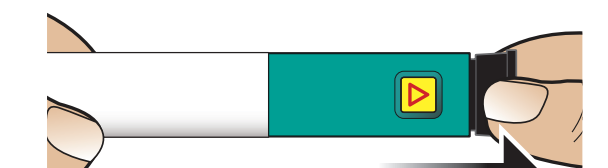
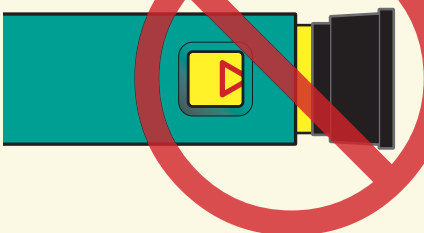
Teriparatide Kamada - Solution for injection in a pre-filled pen

Troubleshooting	
PROBLEM	SOLUTION
A	<p>The yellow shaft is still showing after I push in the black injection button. How do I reset my Teriparatide Kamada pen?</p> 
B	<p>How can I tell if my Teriparatide Kamada pen works?</p> <p>The black injection button should be all the way in to show that the full dose of medicine has been injected from the Teriparatide Kamada pen. Use a new needle every time you inject to be sure your Teriparatide Kamada pen will work properly.</p>
C	<p>I see an air bubble in my Teriparatide Kamada pen.</p> <p>A small air bubble will not affect your dose and it will not harm you. You can continue to take your dose as usual.</p>
D	<p>I cannot get the needle off.</p> <p>1. Put the large outer needle cover on the needle. 2. Use the large outer needle cover to unscrew the needle. 3. Unscrew the needle all the way by giving the large outer needle cover 3 to 5 counter-clockwise turns. 4. If you still cannot get the needle off, ask someone to help you.</p>
E	<p>What should I do if I have difficulty pulling out the black injection button?</p> <p>Change to a new Teriparatide Kamada pen to take your dose as instructed by your physician. When the black injection button becomes hard to pull out, this means there is not enough medication in your Teriparatide Kamada pen for another dose. You may still see some medication left in the cartridge.</p>

CLEANING AND STORAGE

Cleaning your Teriparatide Kamada pen

- Wipe the outside of the Teriparatide Kamada pen with a damp cloth.
- Do not place the Teriparatide Kamada pen in water, or wash or clean it with any liquid.

Storing your Teriparatide Kamada pen

- After each use, refrigerate the Teriparatide Kamada pen right away. Read and follow the instructions in the patient leaflet in section "How should I store Teriparatide Kamada?".
- DO NOT** store the Teriparatide Kamada pen with a needle attached. Doing this may cause air bubbles to form in the medicine cartridge.
- Store the Teriparatide Kamada pen with the white cap on.
- DO NOT** freeze Teriparatide Kamada pen. If the Teriparatide Kamada pen has been frozen, throw the pen away and use a new Teriparatide Kamada pen. If the Teriparatide Kamada pen has been left out of the refrigerator, do not throw the pen away. Place the pen back in the refrigerator and contact your physician or pharmacist.

OTHER IMPORTANT NOTES

- The Teriparatide Kamada pen contains 28 days of medicine.
- DO NOT** transfer the medication to a syringe. This may result in you taking the wrong dose of medicine.
- Read and follow the instructions in the User Manual so that you use your Teriparatide Kamada pen the right way.
- Check the Teriparatide Kamada pen label to make sure you have the right medicine and that it has not expired.
- DO NOT** use the Teriparatide Kamada pen if it looks damaged. Look at the medication in the cartridge. If the medication is not clear and colorless, or if it has particles, do not use it. Contact your physician or pharmacist if you notice any of these.
- Use a new needle for each injection.
- During injection, you may hear one or more clicks - this is normal.
- The Teriparatide Kamada pen is not recommended for use by the blind or by those who have vision problems without help from a person trained in the proper use of the pen.
- Keep your Teriparatide Kamada pen and needles out of the reach and sight of children.

DISPOSAL OF PEN NEEDLES AND PEN

Disposal of pen needles and the Teriparatide Kamada pen

- Before throwing away the Teriparatide Kamada pen, be sure to remove the pen needle.
- Throw away your Teriparatide Kamada pen and used needles as instructed by your physician, local or state laws, or institutional policies.

<p>Dispose of the Teriparatide Kamada pen 28 days after first use.</p>	<p>Date of 1st Use:</p>
<p>Throw Away After:</p>	

CONTACT INFORMATION

If you have questions, need assistance or more information regarding your Teriparatide Kamada pen, contact your your physician or pharmacist.

Manufactured by: Alvogin, Inc., Morristown, NJ, USA

License holder: Kamada Ltd.

The drug registration number in the National Drug Registry in the Ministry of Health: 170-21-36517

Revised in April 2023 according to MOHs guidelines.

3200004076-01