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
Teriparatide Teva Pen  
User Manual

Important: First read the package leaflet that comes inside your Teriparatide carton.

Before you use your Teriparatide Teva Pen, please read the entire User Manual. Follow the directions carefully when using the pen.

The Teriparatide Teva Pen contains 28 days of medicine. Throw away the pen after 28 days, even if it is not completely empty. Do not inject more than one dose in the same day.

Do not transfer Teriparatide to a syringe.

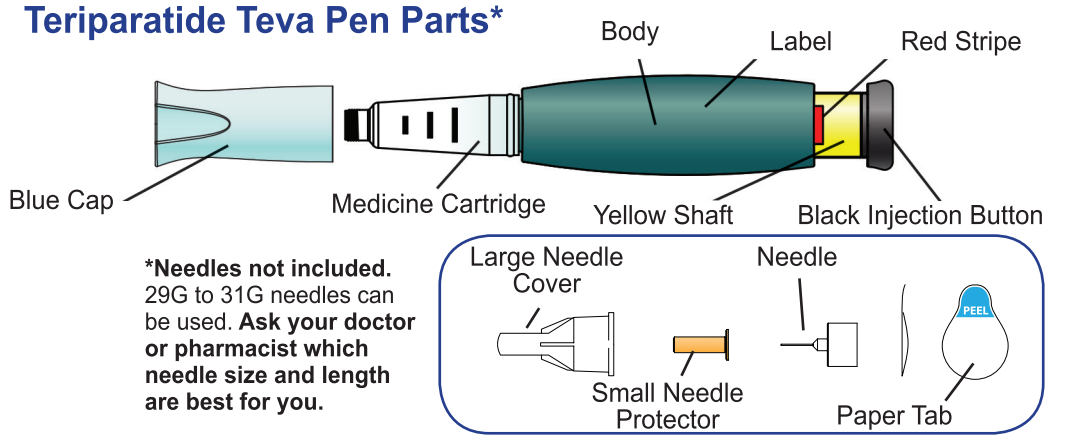Wash your hands before every injection. Prepare the injection site as your doctor or pharmacist instructed.

- 1 Pull off blue cap**  
Check the Teriparatide Teva Pen label to make sure you have the right medicine and that it has not expired. Do not use the pen if the pen looks damaged, the medicine in the cartridge is not clear and colorless, or if it has particles in it.
- 2 Attach new needle**  
Pull off paper tab. Push needle straight onto medicine cartridge. Screw on needle clockwise until firmly attached. Pull off large needle cover and save it.
- 3 Set dose**  
Pull out black injection button until it stops. If you cannot pull out the black injection button see Troubleshooting, Problem E, in this manual. Check to make sure red stripe shows. Pull off small needle protector and throw away.
- 4 Inject dose**  
Gently hold a fold of skin on your thigh or abdomen and insert needle straight into skin. Push in black injection button until it stops. Hold it in and count to 5 slowly. You must wait until the count of 5 to make sure you receive the correct dose. Then pull the needle from skin.
- 5 Confirm dose**  
After injecting the dose: Once the needle is removed from the skin, take your thumb off the black injection button. Check to make sure the black injection button is all the way in. If the yellow shaft does not show, you have finished the injection steps the right way. You should not see any of the yellow shaft. If you do and have already injected the medicine, do not inject yourself a second time on the same day. Instead, you MUST reset the Teriparatide Teva Pen (see Troubleshooting, Problem A, in this manual).
- 6 Remove needle**  
Put large needle cover on needle. Do not try to pull the needle cover back on with your hands. Unscrew the covered needle all the way by giving the large needle cover 3 to 5 counter-clockwise turns. Pull off needle and throw away as directed by your doctor or pharmacist. Push blue cap back on. Right after use, place Teriparatide Teva Pen in the refrigerator.

For more information, or if you have any questions, see the section Troubleshooting in this manual.

Troubleshooting

Problem	Solution
A. The yellow shaft is still showing after I push in the black injection button. How do I reset my Teriparatide Teva Pen?	To reset the Teriparatide Teva Pen, follow the steps below. 1. If you have already injected, DO NOT inject yourself a second time on the same day. 2. Remove the needle. 3. Attach a new needle, pull off the large needle cover and save it. 4. Pull out the black injection button until it stops. Check to make sure the red stripe shows. 5. Pull off the small needle protector and throw it away. 6. Point the needle down into an empty container. Push in the black injection button until it stops. Hold it in and slowly count to five. You may see a small stream or drop of fluid. When you have finished, the black injection button should be all the way in. 7. If you still see the yellow shaft showing, contact your doctor or pharmacist. 8. Put the large needle cover on the needle. Unscrew the needle all the way by giving the needle cover 3 to 5 counter-clockwise turns. Pull off the covered needle and throw away as instructed by your doctor or pharmacist. Push the blue cap back on, and put your Teriparatide Teva Pen in the refrigerator. You can prevent this problem by always using a NEW needle for each injection, and by pushing the black injection button all the way in and slowly counting to five.
B. How can I tell if my Teriparatide Teva Pen works properly?	The black injection button should be pushed all the way in to show that the full dose of medicine has been injected from the Teriparatide Teva Pen. Use a new needle every time you inject to be sure your Teriparatide Teva Pen will work properly.
C. I can see an air bubble in my Teriparatide Teva Pen.	A small air bubble will not affect your dose and it will not harm you. You can continue to take your dose as usual.
D. I cannot get the needle off.	1. Put the large needle cover on the needle. 2. Use the large needle cover to unscrew the needle. 3. Unscrew the needle all the way by giving the large needle cover 3 to 5 counter-clockwise turns. 4. If you still cannot get the needle off, ask someone to help you.
E. What should I do if I have difficulty pulling out the black injection button?	Change to a new Teriparatide Teva Pen to take your dose as instructed by your doctor. When the black injection button becomes hard to pull out, this means there is not enough medicine in your Teriparatide Teva Pen for another dose. You may still see some medicine left in the cartridge.

- Cleaning and Storage:
- Cleaning Your Teriparatide Teva Pen:  
• Wipe the outside of the Teriparatide Teva Pen with a damp cloth.  
• Do not place the Teriparatide Teva Pen in water, or wash or clean it with any liquid.
- Storing Your Teriparatide Teva Pen:  
• After each use, refrigerate the Teriparatide Teva Pen right away. Read and follow the storage instructions in the patient leaflet section 5 - "How to store the medicine?".  
• Do not store the Teriparatide Teva Pen with the needle attached. Doing this may cause air bubbles to form in the medicine cartridge.  
• Store the Teriparatide Teva Pen with the blue cap on.  
• Do not freeze Teriparatide Teva Pen. If the pen has been frozen, throw it away and use a new pen.  
• If the Teriparatide Teva Pen has been left out of the refrigerator, do not throw the pen away. Place the pen back in the refrigerator and contact your doctor or pharmacist.

- Other Important Notes:
- The Teriparatide Teva Pen contains 28 days of medicine.
  - Do not transfer the medicine to a syringe. This may result in you taking the wrong dose of medicine.
  - Read and follow the instructions in this manual so that you use your Teriparatide Teva Pen the right way.
  - Check the Teriparatide Teva Pen label to make sure you have the right medicine and that it has not expired.